

PLAYWORK Ltd. Yasha Hefetz St. 11 Tel Aviv, Israel Tel: +972-50-9300900 | info@playwork.me

Tuesday, December 19, 2023



PlayBall For ASD

April2023



Playwork - Intelligent IOT technology for common physio products Page 1

CONFIDENTIAL - DO NOT DISSEMINATE. This document contains confidential, trade-secret information and is shared only with the understanding that you will not share its contents or ideas with third parties without the express written consent of the plan author.



PLAYWORK Ltd. Yasha Hefetz St. 11 Tel Aviv, Israel Tel: +972-50-9300900 | info@playwork.me

PlayBall: use recommendations for ASD

Playball is an interactive therapy ball that allows patients to play interactive games while performing a variety of interactive ball exercises. The device is specially recommended by therapists for using by kids with ASD and for practicing various skills like: Coordination (uni & bilateral), Visual spatial orientation and perception, visual tracking, increase strength, behavioral skills, inhibition and attention span. As a motivational tool it can also be used for behavioral modification. PlayBall is used almost at any relevant settings including schools, recreational, therapeutic (both in\outpatient) and at home.

Following are general recommendation for using the PlayBall games for various ASD conditions (high\medium\low function), ages (5-21) and practice goal:

1. Game 1: 'PlayMove':

Age\function level	Level 1 (high function)	Level 2 (medium function)	Level 3 (low function
5-8	+	+	-
9-13	+	+	-
14-18	+	+	-
19-21	+	+	+**



** practice with therapist (1 on 1)

Goal: coordination, increase strength, attention span, fine motor

2. Game 2: 'PlayForce':

Age) function lovel	Level 1 (high	Level 2 (medium	Level 3 (low
Age\function level	function)	function)	function
5-8	+	-	-
9-13	+	-	-
14-18	+	+	-
19-21	+	+	-



Goal: Bilateral coordination, visual spatial orientation & perception, increase strength.

Playwork - Intelligent IOT technology for common physio products Page 2

CONFIDENTIAL - DO NOT DISSEMINATE. This document contains confidential, trade-secret information and is shared only with the understanding that you will not share its contents or ideas with third parties without the express written consent of the plan author.



PLAYWORK Ltd. Yasha Hefetz St. 11 Tel Aviv, Israel Tel: +972-50-9300900 | info@playwork.me

3. Game 3: 'KnifeHit':


Age\function level	Level 1 (high function)	Level 2 (medium function)	Level 3 (low function
5-8	-	-	-
9-13	+	-	-
14-18	+	+	-
19-21	+	+	-



Goal: Bilateral coordination, inhibitory control, visual spatial orientation & perception, increase strength

4. Game 4: 'Bottle vr. Bulet':

Age\function level	Level 1 (high function)	Level 2 (medium function)	Level 3 (low function
5-8	-	-	-
9-13	+	-	-
14-18	+	+	-
19-21	+	+	-

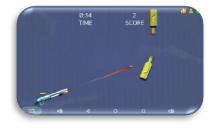

Goal: Visual orientation & perception, visual tracking, attention span

5. Game 5: 'Flying Rocket':

Age function lovel	Level 1 (high	Level 2 (medium	Level 3 (low
Age\function level	function)	function)	function
5-8	-	-	-
9-13	+	-	-
14-18	+	-	-
19-21	+	-	-



Goal: High function\multi task.

Important! PlayBall can be used for various common ball exercises according to therapist's supervision and guidance. The above mentioned are general recommendations. For operating with any specific user\patient, the way of using the device should be modified, instructed and supervised by a therapist (OT\PT or other certified ASD therapist).

Playwork - Intelligent IOT technology for common physio products Page 3

CONFIDENTIAL - DO NOT DISSEMINATE. This document contains confidential, trade-secret information and is shared only with the understanding that you will not share its contents or ideas with third parties without the express written consent of the plan author.